# Combipress N/LM Kaltpolymerisat

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Stand der Information 2018-11

# Combipress N/LM Cold-curing polymer

## (en) Instructions for use, please read carefully!

### Intended use

For producing prostheses, especially for combination and hybrid dentures, as well as for relining and repairing prostheses.

### Indication

- Total or partial tooth loss

### Contraindication

- Hypersensitivity to one of the components

### Composition

Combipress N/LM powder contains

- PMMA (polymer and copolymer)
- barbituric acid catalyst system
- organic colorants
- inorganic pigments

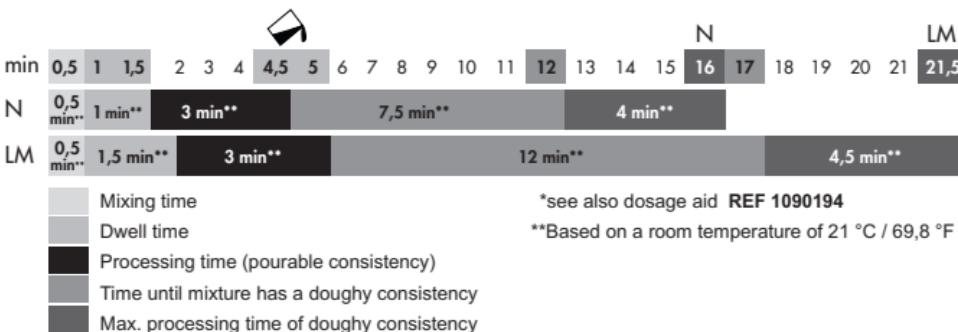
Combipress N/LM liquid contains

- MMA (methylmethacrylate)
- dimethacrylate
- barbituric acid catalyst system

### Dosage and application

- See table for mixing ratios and processing times.
- Use class IV plaster for making casts where possible.
- Always wet casts sufficiently for 5 to 15 minutes depending on the degree of dryness.
- Use standard alginate isolation to separate plaster from acrylic.
- Do not compress the powder when using dosage aids.

Times and quantities	Combipress N	Combipress LM
Mixing ratio*	5 g : 3,5 ml 10 g : 7 ml 20 g : 14 ml 30 g : 21 ml	
Polymerisation temperature	45 °C / 133 °F	
Polymerisation time	10 min	15 min
Pressure required during polymerisation	2 - 2,5 bar	

## **Processing the acrylic**

To reduce the residual monomer content, the finished (sanded) but not polished polymerised workpiece should be stored in water at room temperature for about 48 hours before delivery.

## **Trimming**

To prevent inaccurate fitting after polymerisation, avoid excessive heat development during trimming and polishing.

## **Bond**

- Ensure that no impurities such as wax or isolating agent are present.
- Roughen the basal surfaces of the acrylic teeth before bonding.
- Mechanical retention is recommended.
- Adequate mechanical retention is essential for ceramic teeth.
- Use artConnect PLUS bonding agent to support bonding.

## **Restoration**

Products of Combipress N/LM can also be restored and supplemented at any time with all standard MMA-based cold-curing polymers.

Following roughening of the surface (sanding / blasting / grinding), further conditioning is recommended, preferably with artConnect PLUS or a similar MMA-based bonding agent.

## **Storage**

- Store in a dry and cool area
- Avoid direct sunlight

## **Notes**

- Close all powder and liquid containers tightly after use.
- Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery.
- Do not use the material after the expiration date.
- Record the LOT number with every process that requires identification of the material.
- The residual monomer content after polymerisation of Combipress N/LM is < 4,5 %.
- Processing frameworks may generate dust which can lead to mechanical irritation of the eyes and airways. Therefore, always ensure that the extraction system on your milling machine and at the workplace is working properly for individual post-processing and that you are using personal protective equipment.

## **Disposal**

- The fully polymerized components of Combipress N/LM are disposed of as residual waste.
- Disposal of nonpolymerized product residues according to legal regulations.
- Do not allow entry into the sewage system or waterways.

## **Danger and safety instruction CombiPress N/LM Monomer**

H225 Highly flammable liquid and vapour.

H315 Causes skin irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P272 Contaminated work clothing should not be allowed out of the workplace.

P262 Do not get in eyes, on skin, or on clothing.

P210 Keep away from heat. No Smoking.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

## **Danger and safety instructions CombiPress Polymer**

H317 May cause an allergic skin reaction.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

## **Classification**

CombiPress N/LM conforms to EN ISO 20795-1 Typ 2 Class 1.

**The product properties are based on compliance with and observation the instructions for use.**

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from [www.merz-dental.de](http://www.merz-dental.de).

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**Merz Dental GmbH**